

Rege-Derm Kit[®] Rege-Derm Patch[®]

Instruction for Use

REGEDERM Patch Finger Cot is used for

Cleaning, scrubbing, chemical or minor trimming/scraping or soft Non excisional debridement to remove fragments of dead tissue.

It can be used also for swabbing and samples collection for laboratory Investigation or culture



Uses of REGEDERM Chitosan 5 MI Vial solution

- 1- After Cleaning the affected site, put 4-5 drops of Chitosan Solution { acidity of Solution is 6.2 PH } over the cleaned area "we prefer to pour the drops one by one over sterile degradable Chitosan Matrix " different sizes are available " leave for 10 Minutes for drying "
- 2- After complete drying, cover with Our Chitosan adhesive dressing " different sizes and shapes are available .
- 3- Leave for 4-5 days and repeat the same procedure if needed .



Medical Supply Ltd. Canada
A Regenerative Medicine Technology Company

www.dcaremedical.ca - info@dcaremedical.ca